

448, 450 Soi Supharaj, Phaholyothin Rd., Samsaen Nai, Phyathai, Bangkok 10400, Thailand Tel: 02 090-2591/4 Ext.229 [Service Dept.] Fax: 02 270-0513 [Service Dept.] เลขประจำตัวผู้เลือกาษี: 0105529003833

Preventive Maintenance

HOSPITAL CONTROL NO.

HOSPITAL NAME	AL NAME DEPARTMENT						
INSTRUMENT	Arctic Sun	MANUFACTURE	Bard	MODEL	AS5000		
SERIAL NUMBER							
DATE		Next Due		1	Period of PM	Months	
Action				Res	ult	Remark	
1. External Check				Pass	Fail		
- Check Cassing							
- Check Screen i	is intact						
- Check Castor (Condition						
- Check power cable condition (Power on the unit)							
2. Function Checl				Pass	Fail		
- Ensuer unit sta							
		nd display has on burnou	t line (Select hypotherm	ia			
•	ature dongle 37°C						
- Drain and repl							
3. Performance C	heck						
- Warming Test				Pass	Fail		
	erature 40 °C, durat	ion : 30 min					
Start therapy							
Check temperature reaches 30 °C within 15 min, flow rate 1.6-1.9 L/min							
- Cooling Test				Pass	Fail		
-	erature 4 °C, duratio	on : 30 min					
Start therapy							
Check temperature reaches 6 °C within 30 min, flow rate 1.6-1.9 L/min							
4. System Calibra				Pass	Fail		
Next calibration		Hour.	Users.				
5. Electrical safet				Pass	Fail		
- Perfoem electric	al Safety analyser te	est					
Overall Test Resul Comments :	lt: PASS / FAIL						
				Tested By: (Signature)_ (Name)			
				(Haille)	Custome	r Service	