

448, 450 Soi Supharaj, Phaholyothin Rd., Samsaen Nai, Phyathai, Bangkok 10400, Thailand Tel: 02 090-2591/4 Ext.229 [Service Dept.] Fax: 02 270-0513 [Service Dept.] เลขประจำตัวผู้เลียภาษี: 0105529003833

## **Preventive Maintenance**

HOSPITAL CONTROL NO.

HOSPITAL NAME		DEF	PARTMENT				
INSTRUMENT Infusion Pump		MANUFACTURE BD		MODEL	Alaris GP		
SERIAL NUMBER							
DATE Next Due						Period of PM	Months
Action				Pass	Fail	Remark	
Check all function i	in PVP Work Flow:						
Enter access code 2	212 and fo to PVP Work Flow	<u>!</u>					
<ul> <li>Overall conditio</li> </ul>	n / Cleaning						
Main Self-Test							
Software Version							
Date/Time							
Serial Number							
Door Frame							
<ul> <li>Selfty Clamp</li> </ul>							
• Sear							
Audio Test							
Chequerboard							
• LED's							
• Keypad							
Nurse Call							
RS232 Loop Back							
	Set rate to zer	o (or lowest valve pos	sible), Clear \	Volume i	nfused and V	ТВІ	
Electrical Safety Test		Test Result are stored :		Pass	Fail	Remark	
- Class I Typf CF		Electronically					
Test in accordance	e whit the standard	Print-out					
EN60601-1 and test equipment operation		Other					
manual.							
Overall Test Resul Comments :	it: PASS / FAIL						
					(Name)		

**Customer Service**